

IPS e.max® SMILE AWARD

2nd PLACE



DR FERRAN LLANSANA
DENTIST, SPAIN



JUAN SAMPOL REUS
DENTAL TECHNICIAN, SPAIN

**"Perfectly handled with ultra thin veneers.
A very positive impact on
the entire face."**

Dr Galip Gürel, Member of the judging panel



Initial situation

This young female patient consulted our practice about a smile makeover. Her teeth showed wide and rounded interincisal angles. Tooth 13 exhibited a slight vestibular inclination. The front teeth lacked canine guidance. Tooth 11 showed a slightly lingual orientation. The aim of the treatment was to optimize and lighten the appearance of the teeth.



Final situation

IPS e.max®



In order to visualize the esthetic improvements proposed by the dental technician, a wax-up was created. With the help of the wax-up, the incisal edges and the new smile together with the correct morphology were evaluated. Subsequently, the teeth were whitened and lightly prepared in order to facilitate the placement of the restorations. The somewhat bulky vestibular portion of tooth 13 was reduced using a minimally invasive approach. The veneers were layered on refractory dies using IPS e.max Ceram Dentin, Incisal and Impulse materials. Due to the outstanding chameleon effect of the veneers, the restorations blended in seamlessly with the natural tooth structure.



Veneers: IPS e.max® Ceram